Handy Cordless  
Prophylaxis Micromotor

# iProphy

OPERATION  
MANUAL

CE 0197 MADE IN JAPAN

OM-E0663E 000

## Contents

<b>1. User and Indications for Use</b> .....	<b>2</b>
<b>2. Precautions for Handling and Operation</b> ..	<b>2</b>
<b>3. Product Description</b> .....	<b>6</b>
3-1 Package contents .....	6
<b>4. Part Names</b> .....	<b>7</b>
4-1 Keys on the Operation Panel .....	7
4-2 Indicators on the Operation Panel .....	8
<b>5. Preparation for Use</b> .....	<b>9</b>
5-1 Connecting the Stand .....	9
5-2 Connecting and disconnecting the contra angle head .....	9
5-3 Inserting and Removing the Cup/Brush .....	10
5-4 Check before treatment .....	11
<b>6. Operation Procedure</b> .....	<b>12</b>
6-1 Charging the Battery .....	12
6-2 Power Supply .....	13
6-3 Setting of Rotation Speed .....	13
6-4 Operation .....	13
6-5 Functions .....	13
6-6 Protection Circuit .....	13
<b>7. Post-use Maintenance</b> .....	<b>14</b>
7-1 Preparation for maintenance .....	14
7-2 Cleaning and disinfection .....	14
7-3 Lubrication .....	18
7-4 Sterilization .....	19
7-5 Storage .....	20
<b>8. Maintenance</b> .....	<b>21</b>
8-1 Changing the battery .....	21
8-2 Mounting and Removing the AC Adaptor Plug Head .....	23
8-3 Periodical Maintenance Checks .....	23
<b>9. Troubleshooting</b> .....	<b>24</b>
9-1 Error code .....	24
9-2 Problems and actions .....	26
<b>10. After-sales Service</b> .....	<b>27</b>
10-1 Warranty .....	27
10-2 Spare parts list .....	27
10-3 Option parts list .....	27
10-4 Disposing product .....	27
<b>11. Specifications</b> .....	<b>28</b>
11-1 Specifications .....	28
11-2 Classification of equipment .....	29
11-3 Operation principle .....	29
11-4 Symbol .....	30
<b>12. EMC Information (Electromagnetic Compatibility Information)</b> .....	<b>31</b>

# 1 User and Indications for Use

User: dentist, hygienist

Indications for use: Removal and polishing of accretions/deposits on the surface of teeth and dental restorations (fillings and prostheses, etc).

# 2 Precautions for Handling and Operation

- Please read these precautions carefully and use only as intended or instructed.
- Safety instructions are intended to avoid potential hazards that could result in personal injury or damage to the device. Safety instructions are classified as follows in accordance with the seriousness of the risk.

Class	Degree of risk
<b>▲ DANGER</b>	Hazard that could result in personal death or serious injury if the safety instructions are not correctly followed.
<b>▲ WARNING</b>	Hazard that could result in serious injury or damage to the device if the safety instructions are not correctly followed.
<b>▲ CAUTION</b>	Hazard that could result in light or moderate injury or damage to the device if the safety instructions are not correctly followed.
<b>NOTICE</b>	General product specification information highlighted to avoid product malfunction and performance reduction.

## ▲ DANGER

- Do not attempt to disassemble the product nor tamper with the mechanism except as recommended by NSK in this operation manual. This may result in an injury, electric shock or fire.
- Do not wet the motor handpiece with water, disinfectant, saline, or other fluids. The protective function of the rechargeable battery may be damaged which may result in overheating, rupture, or ignition due to the abnormal current or voltage to the battery.
- Do not charge, use or store the product in direct sunlight, a vehicle, the vicinity of fire and heater, or other location where temperatures may be outside the range shown in the specification table in this manual. This may result in discoloration, overheating, smoking, reduction in performance or reduction in the life of the rechargeable battery.
- Do not use AC adaptor other than those supplied by NSK. Use of other cords may result in electric shock, fire or breakdown. Use the accessories that are specified by NSK.
- Do not heat the motor handpiece or throw the motor handpiece into the fire. This may cause overheating, smoking, rupture, or ignition due to melting of the insulator in the internal battery, damage to the gas relief valve or safety mechanism, or ignition of the battery fluid.
- Do not pierce the motor handpiece, hit it with a hammer, or step on it. This may cause overheating, smoking, rupture, or explosion due to an internal short-circuit as a result of rupture or deformation of the internal battery.
- If you find battery fluid, deformation, breakage or partial discoloration of the motor handpiece casing, stop use immediately and contact your authorized NSK dealer. Continued use may result in electric shock or fire.
- If the leaking battery fluid gets into your eyes, wash eyes thoroughly with clean water immediately and see your doctor. Failure to do so may result in loss of sight.

**▲ DANGER**

- If the battery leaks or a strange smell comes from the motor handpiece, stop use immediately and keep the motor handpiece away from fire and other combustible materials. This may cause smoking, rupture, or ignition due to battery fluid catching fire.
- Do not place the motor handpiece or the internal battery pack in a microwave oven or a high-pressure container. Rapid heating and a breakage in a sealed condition may cause heat generation, rupture or ignition.

**▲ WARNING**

- Do not operate the product close to patients with cardiac pacemakers as there is a danger that may affect the pacemaker.
- If either the dental practitioner or the patient has a previous history of allergy, see also the accompanying documents of the cup and the brush.
- Do not handle the AC adaptor and motor cord with wet hands. This may cause electric shock.
- Keep away from explosive substances and flammable materials. Do not use on a patient who has been administered Nitrous Oxide and in the vicinity of such a patient.
- If there is an abnormality such as smoke or smell of burning plastic from the product, turn off the power immediately, unplug the power cord from the outlet, and contact your authorized NSK dealer. Continued use may result in an electric shock or fire.
- The system may present a possibility of malfunction when used in the presence of an electromagnetic interference wave. Do not install the system in the vicinity of any device which emits magnetic waves. Turn off the Main Power Switch of the system if an ultrasonic oscillation device or an electrode knife is used close to the product.
- Do not spill water, saline or a disinfectant on to the motor handpiece and the stand. This may result in an injury, electric shock or fire.
- If the battery fluid leaks and adheres to skin or clothing, wash the exposed skin thoroughly with clean water immediately and completely wash away the fluid. Failure to do so may result in skin eruption.
- If you do not use this product for a long period, remove the battery to avoid the battery liquid leaking.
- Do not use AC adaptor, other than those specified by the NSK. This may result in performance degradation of electromagnetic compatibility (EMC), such as increased EMISSIONS\*1 or decreased IMMUNITY\*2, of the product.

\*1: Electric noise (Mainly electromagnetic wave)

\*2: Not to be obstructed by the electromagnetic wave generated by the electrical equipment.

- After treatment, the contra angle head should be cleaned, lubricated and sterilized immediately, and then store it. Failure to properly maintain the contra angle head may cause malfunction, overheating leading to burns. (See "7 Post-use Maintenance")
- Keep any debris or other foreign materials away from the inside of the gear of the contra angle head and the motor handpiece. Foreign materials remaining inside may lead to overheating, causing burn injuries or other accidents.
- Ball bearings are consumables and may require replacement. Be sure to inspect the contra angle head before use (See "5-4 Check before treatment"). If abnormal vibration or noise are found, the bearings or other internal parts may be worn out or damaged. Damaged parts may lead to contra angle head overheating causing burns. If overheating is suspected, stop using the contra angle head immediately and contact your authorized NSK dealer.

## Precautions for Handling and Operation

### ⚠ CAUTION

- When operating the product, always consider the safety of the patient.
- Do not use other than by the users or for the intended uses noted in this manual.
- The end user shall be responsible for any judgment that relates to the application of this product to a patient.
- Users are responsible for the operational control, maintenance and continual inspection of this product.
- This device is for indoor use only.
- Keep the stand on a level surface.
- Do not allow any impact on to the product. Dropping may result in an injury and may cause overheating or fire due to failure of the internal component.
- Operators and all others in the area must wear eye protection, mask, and gloves when operating the product.
- Do not use or charge the motor handpiece on or under a flammable material, or being covered. This may cause overheating, rupture or fire.
- Do not use the motor handpiece where strong static electricity is generated as its protection mechanism may be damaged or the liquid of its battery pack may leak.
- The maximum surface temperature of the motor handpiece might reach 45°C depending on the situation in which the system is used. Leave it until it cools sufficiently.
- Do not use this product at an unnatural angle or with excessive force. This may cause the product to break or damage the lining of the mouth.
- If the product functions abnormally during use, stop using the product immediately and contact your authorized NSK dealer.
- Turn OFF the power switch after use. If you do not use this product for a long period, remove the AC adaptor from the outlet.
- Install the device so that the AC adaptor can be easily pulled out of the power outlet. Leave space within 15 cm of the AC adaptor.
- Do not use the following fluids to wipe, immerse or clean the product; electrolyzed oxidizing water (strong/super acid water), strong acid/alkaline chemicals, solvents containing chlorine, solvents such as benzene or thinner. (See “5–4 Check before treatment”).
- Do not lubricate the motor handpiece. This may cause the motor handpiece to heat up or cause its malfunction.
- The contra angle head is delivered in a non-sterile condition and must be sterilized before use.
- Perform regular function and maintenance checks (See “8–3 Periodical Maintenance Checks”).
- If the motor handpiece has not been used for a long period, check that it functions correctly before using it on a patient.
- It is recommended that a spare be kept on hand in case of a breakdown during treatment.
- iProphy needs special precautions regarding EMC and needs to be installed and put into service according to the “12 EMC Information (Electromagnetic Compatibility Information)”.
- Portable and mobile RF communications equipment can affect iProphy. Do not use RF communications equipment near the iProphy.

Intermittent operation	
ON	OFF
3 min	5 min

### **⚠ CAUTION**


- The use of ACCESSORIES, transducers and cables other than those specified, with the exception of transducers and cables sold by the manufacturer of iProphy as replacement parts for internal components, may result in increased EMISSIONS or decreased IMMUNITY of iProphy.
- iProphy should not be used adjacent to or stacked with other equipment and that if adjacent or stacked use is necessary, iProphy should be observed to verify normal operation in the configuration in which it will be used.

### **NOTICE**

- If the product needs repair, contact your authorized NSK dealer. Service personnel will repair the product according to the service manual.
- No special training is required for this device.
- Using the motor handpiece may affect computers and LAN cables in the vicinity of the motor handpiece. This causes noise in radio receivers nearby.

### 3 Product Description

#### 3-1 Package contents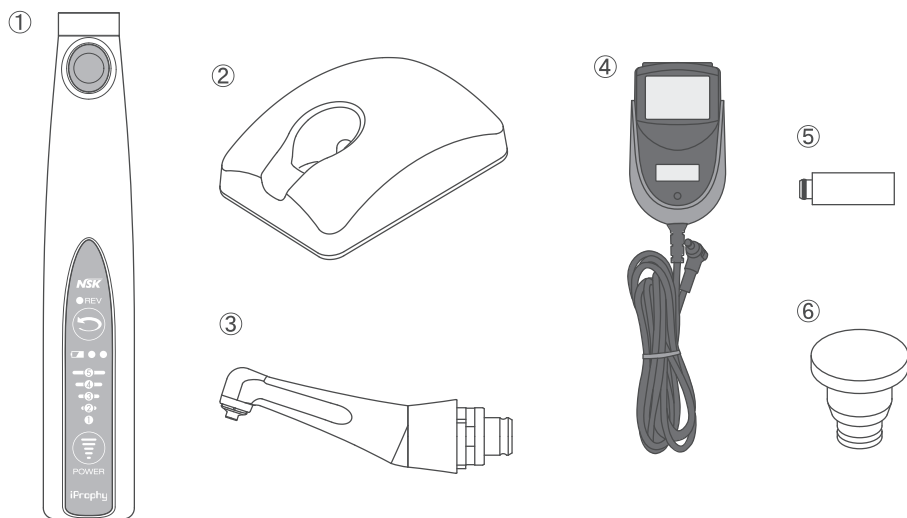


iProphy complete set

No.	Part Name	Quantity
①	iProphy Motor handpiece	1
②	iProphy Stand	1
③	iFX57 (Contra angle head)	1
④	AC ADAPTOR	1
⑤	F-Type Spray Nozzle	1
⑥	MOTOR CAP	1

There is also a set without contra angle head.

## 4 Part Names

### ■ Motor Handpiece

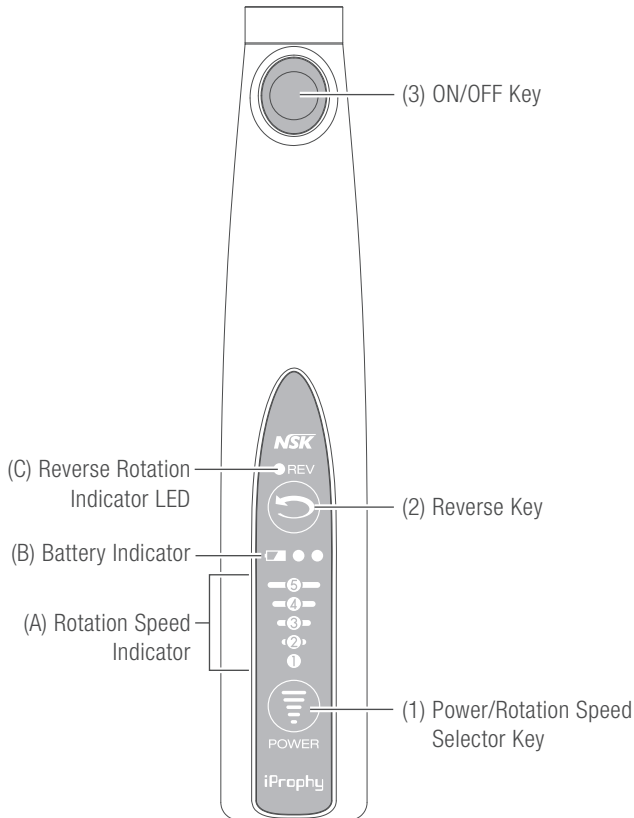### 4-1 Keys on the Operation Panel

- (1) Power/Rotation Speed Selector Key  
This key is used for turning ON / OFF the power. Press and hold it for 1 second or longer to turn ON/OFF.  
When this key is pressed briefly while the power is ON, the rotation speed changes. Each time it is pressed, the speed increases by one level. Once it has reached the maximum level, it returns to the minimum level. There are 5 speed levels.
- (2) Reverse Key  
Press this key while the Handpiece is not rotating to switch the set rotational direction between the forward rotation (clockwise rotation) and the reverse rotation (anticlockwise rotation). If this key is pressed while the Handpiece is rotating, the direction of the rotation does not change.
- (3) ON/OFF Key  
This key is used for turning ON/OFF the rotation of the motor handpiece. When the key is pressed for 1 second or longer, the motor handpiece rotates as long as this Key is held down. When the key is released, the rotation stops.

**4-2 Indicators on the Operation Panel**

(A) Rotation Speed Indicator

The rotation speed is displayed in LED.

-  2,500min<sup>-1</sup>
-  2,000min<sup>-1</sup>
-  1,500min<sup>-1</sup>
-  1,000min<sup>-1</sup>
-  500min<sup>-1</sup>

(B) Battery Indicator

The current battery charge level is displayed. The LED lights or flashes while charging.

2 LED lighting : Full charge or nearly full charge

1 LED lighting : About 30-80% remains

1 LED flashing: Less than about 30% remains

(C) Reverse Rotation Indicator LED

This light is on when the reverse rotation (anticlockwise) is set.

## 5 Preparation for Use

### ⚠ CAUTION


- The accompanying AC adaptor should be used for this product only. Do not use it for anything else.
- When pulling out the AC adaptor, hold the plug. If the cord of the AC adaptor is pulled, the wire inside may be broken.

### NOTICE

If you start charging while the motor handpiece is rotating, the rotation stops and the charging starts.


### 5-1 Connecting the Stand

Insert the AC adaptor plug into the connector on the back of the Stand until it stops. When disconnecting the AC adaptor plug, hold the Stand firmly and pull it out straight.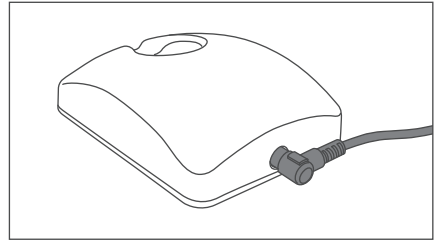


### 5-2 Connecting and disconnecting the contra angle head

■ Attaching the barrier sleeve to the motor handpiece  
Attach the disposable barrier sleeve to the motor handpiece and secure the end of the barrier sleeve with the barrier film.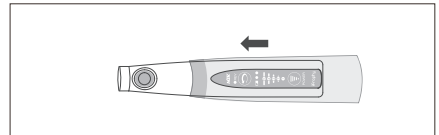

■ Connecting the contra angle head

- 1 Make sure that the power is OFF.
- 2 Push the contra angle head till its alignment pins click into the alignment slots in the Motor Handpiece. The contra angle head can be mounted at 6 patterns of angles.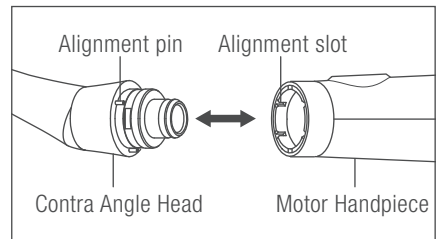


### ⚠ CAUTION

- Use a contra angle head that has been sterilized.

## Preparation for Use

■ Attach the barrier sleeve to the contra angle head  
Attach a disposable barrier sleeve to the contra angle head and secure the end of the barrier sleeve with the barrier film.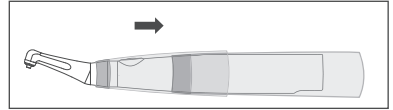

■ Disconnecting the contra angle head

- 1 Turn OFF the motor handpiece.
- 2 When removing the contra angle head, pull it out axially.


### ⚠ CAUTION

- Before connecting and disconnecting the contra angle head, turn off the power to prevent unintentional rotation of the motor handpiece.
- When connecting or disconnecting the contra angle head, do not twist nor turn the contra angle head.
- Do not connect the contra angle head other than those specified by NSK (See “10-3 Option parts list”).

## 5-3 Inserting and Removing the Cup/Brush

■ Inserting

- 1 Insert the screw shank of the cup or brush into the screw hole of the contra angle head.
- 2 Rotate the motor handpiece at a low speed clockwise to fix the screw shank.
- 3 Check that the screw part of the cup or brush is inserted to the end.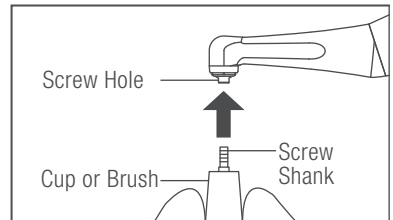

■ Removing

Hold the cup or brush and operate the motor handpiece in the reverse rotation at a low speed.

**▲ CAUTION**


- When inserting the cup or brush, use your fingers to lightly screw in the cup or brush. Then run the motor handpiece at a low speed in a clockwise direction to secure the screw. If the motor handpiece is running prior to screwing the cup or brush into position with your finger the screw shank may not be properly screwed into the hole, which causes damage to the contra angle head.
- Do not use the cup or brush with problems listed below. This may cause the cup or brush, to break, to come off, or to be difficult to come off.
  - Screw shank of the cup or brush that are bent, deformed, worn, rusted, broken, deficient.
  - The cup or brush that have been tampered with.
- Use a cup or brush with screw shank compatible with contra angle head.
- Always keep the screw shank of cup or brush clean. Dirt or debris in the inside of the rotating shaft could cause axial runout.
- Make sure that the screw shank of the cup or brush is inserted to the end.
- For usage method of the cup or brush, see the operation manuals issued by manufacturer.
- Do not exceed the drive motor speed recommended by the cup or brush manufacturer. Be sure to use the motor handpiece in the forward rotation during treatment. If used in reverse rotation, the cup and brush may come off.

**5-4 Check before treatment**

Before use, rotate the motor handpiece outside the oral cavity of the patient to check that nothing is wrong.

If you notice anything unusual when checking, such as unsteadiness, vibrations, abnormal sounds, or overheating, immediately stop using the motor handpiece, and contact your authorized NSK dealer.


- 1** Connecting the contra angle head to the motor handpiece (See “5-2 Connecting and disconnecting the contra angle head”).
- 2** Insert the screw shank of the cup or brush into the screw hole of the contra angle head (“5-3 Inserting and Removing the Cup/Brush”).
- 3** Operate the motor handpiece within allowable rotating speed range of the cup or brush for 1 minute and check for abnormal vibration, or abnormal noise (“6-4 Operation”).
- 4** Stop the motor handpiece and check for abnormal heating of the contra angle head surface.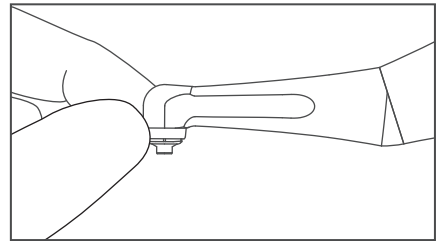

## 6 Operation Procedure

### 6-1 Charging the Battery

- 1 Firmly insert the AC adaptor into an outlet.
- 2 Set the motor handpiece on to the stand.
- 3 An LED light in the battery indicator starts blinking as charging starts.
- 4 When two LED lights in the battery indicator are on, charging is complete.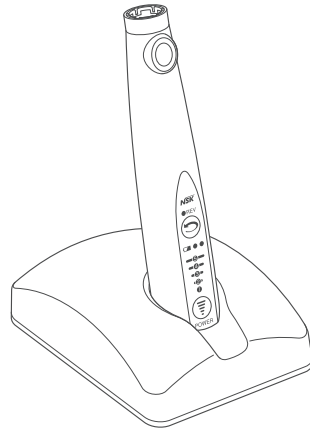

### ⚠ CAUTION

- Insert the AC adaptor into an outlet matching specifications. Failure to do so may cause a malfunction.
- When setting the motor handpiece to the stand, do not allow any hard impact to the motor handpiece. It may cause damage to the motor handpiece and stand.
- When setting the motor handpiece to the stand, pay attention to the direction. Setting it in the wrong direction will cause damage to the motor handpiece and the motor handpiece to fall.
- Wipe liquid off the motor handpiece before setting the motor handpiece to the stand. If the charging terminal gets dirty, rust may occur and charging cannot be performed.
- Do not place conductive parts (eg. wire or safety pins) on the charging terminal part of the stand.
- The Stand is exclusively for the motor handpiece of this product. Never use the Stand for any other purpose.
- The standard charging time is approximately 1.5 hours. This may vary depending on the battery condition, age, temperature and other factors. With older batteries the charging time and operating time may become significantly shorter.
- If the device will not be used for a long period of time, charge the battery before use. If not charged periodically, the battery will become fully discharged and will not charge properly. If unable to charge the battery replace it with a new one.
- During charging, the battery temperature is measured. The battery cannot be charged properly in an environment where temperature changes rapidly (such as near windows exposed to direct sunlight and an air outlet of a fan heater). Charge the battery where the temperature does not change very much.
- Battery charge may not start in the following cases:
  - The battery temperature is too low or too high (approx. 0°C or less. 40°C or more)
  - The battery voltage is enough
  - The battery voltage is abnormal

**NOTICE**

- Generally rechargeable batteries constantly discharge little by little. We recommended you charge the battery just before use.
- This product is turned off automatically when voltage reduction in the battery pack is detected. Immediately charge the battery. When the product is turned on again after being left for some time, voltage reduction is sometimes not detected. This is due to battery characteristics and not faulty. Voltage reduction and remaining capacity of the battery pack do not necessarily correspond to each other. It simply provides an indication.

**6-2 Power Supply**

To turn ON/OFF power, press Power/Rotation Speed Selector Key for one second or longer. When the power is turned ON, the current setting is displayed on the LED light.

**6-3 Setting of Rotation Speed**

To adjust the rotation speed, press the Power/Rotation Speed Selector Key. The setting can also be changed during rotation.

**6-4 Operation**

- 1 Press the ON/OFF Key briefly to start the rotation of the motor handpiece. Press again to stop the rotation.  
If pressing the ON/OFF key for 1 second or longer, the motor handpiece will rotate pressing the ON/OFF key, and will stop when releasing it. (Temporary rotation operation)  
When reverse rotation is set, the motor handpiece rotates while the ON/OFF key is kept pressed, and stops when the key is released.
- 2 Pressing the Power/Rotation Speed Selector Key for 1 second or longer to turn OFF the power after treatment.

**6-5 Functions**

- 1 Auto-Power OFF Function  
When the ON/OFF Key has not been operated for approximately 10 minutes, the power is turned off automatically to save energy and prevent accidents. If the motor handpiece is rotating, the power is not turned off even if the ON/OFF key is not operated.
- 2 Last Memory Function  
The settings when the power is turned off remains until the next power on.

**6-6 Protection Circuit**

When the product is operated under a load exceeding the allowable limit the protection circuit operates to stop the motor and prevent a hazard or damage. The error LED is displayed on the operation panel (See "9-2 Problems and actions") .

## 7 Post-use Maintenance

Maintain the product after each patient treatment as follows.

### 7-1 Preparation for maintenance

#### ⚠ WARNING

- Maintain the contra angle head properly as follows. Improper maintenance may cause malfunction, and burns due to overheating:
  - Be sure to clean the contra angle head immediately (within 30 minutes) after use to remove residue.
  - Be sure to disinfect the contra angle head, within 2 hours after cleaning.

#### ⚠ CAUTION

- Do not clean, immerse, or wipe the product with electrolyzed-oxidizing water (strong acidic water, super acidic water), strong acidic and strong alkaline liquid solution, solvent containing chlorine, solvents such as benzene or thinner.
- Follow any additional local directives, standards, and guidelines for cleaning, disinfection, lubrication and sterilization.

- 1 Wear eye protection glasses, a mask, and gloves to prevent infection.
- 2 Turn off the motor handpiece.

#### NOTICE


- To prevent dirt or liquid entering the motor handpiece, attach the motor cap to the motor handpiece when the contra angle head is not connected.

### 7-2 Cleaning and disinfection

#### [For contra angle head]

[Cleaning of Casing]

- 1 Remove the cup or brush from the contra angle head.
- 2 Remove all debris on the surface of the product with wipes (Minuten Wipes:ALPRO).
- 3 Remove the disposable barrier sleeve from the contra angle head and the motor handpiece.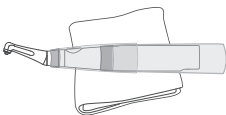

- 4 Remove the contra angle head from the motor handpiece.

- 5** Wipe the debris on the surface of the contra angle head with a cloth moistened with water.


- 6** Clean the surface of the contra angle head under running water with a soft bristled brush for 20 seconds or longer.

Water temperature:  $\leq 38^{\circ}\text{C}$

Water quality: Water available as drinking water

Water amount:  $\geq 3.5$  L/min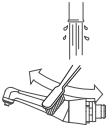

- 7** Clean the rotation shaft under running water with a interdental brush (ISO 16409 size 4 (wire thickness 0.7mm)) for 20 seconds or longer.

Water temperature:  $\leq 38^{\circ}\text{C}$

Water quality: Water available as drinking water


Water amount:  $\geq 3.5$  L/min

- 8** Wipe the residual moisture on the surface of the contra angle head with a dry cloth.

- 9** After cleaning, under appropriate lighting ( $\geq 500\text{lx}$ ), inspect the contra angle head for contamination. If any visible contamination remains, repeat the process from the procedure No. 6 until it is visually clean.

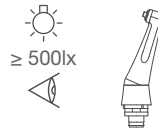## Post-use Maintenance


[Cleaning and Disinfection of Interior]

### ⚠ CAUTION


- Before cleaning and disinfecting the interior, be sure to clean the casing.

Adaptor (ADS 2: Miele/63500: Melag) is to be prepared by a user.

- 1** Wipe the contact part of the contra angle head and adaptor (ADS 2: Miele/63500: Melag) with disinfectant wipes (Minuten Wipes: ALPRO).

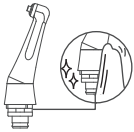
- 2** Attach the contra angle head to the adaptor (ADS 2: Miele/63500: Melag) of the washer-disinfector.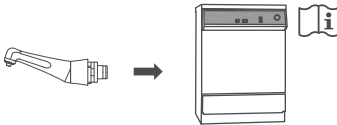

- 3** Clean and disinfect the contra angle head.
- 4** Remove the contra angle head from the adaptor (ADS 2: Miele/63500: Melag) of the washer-disinfector.
- 5** Wipe the residual moisture of the contra angle head with a dry cloth or blow it off with filtered clean compressed air ( $\leq 0.35\text{MPa}$ ) until there is no moisture in the interior and exterior of the contra angle head.


- 6** Under appropriate lighting ( $\geq 500\text{lx}$ ), inspect the contra angle head for contamination. If any visible contamination remains, repeat the process from procedure No. 1 until it is visually clean.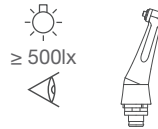

- 7** Proceed to the section “7–3 Lubrication” and “7–4 Sterilization”.

**⚠ CAUTION**

- If paste adheres to the thread of the contra-angle head rotating shaft, we recommend cleaning only the head end with an ultrasonic washer. Be sure to lubricate after cleaning.
- Use a cleaning and disinfecting device (washer-disinfector) complying with DIN EN ISO 15883 (e.g. Miele washer-disinfector G7781 / G7881; Melag Melatherm), operating with a maximum pH value of 10.5 (e.g. neodisher, Dr. Weigert) and equipped with an appropriate adaptor. Proof of suitability for the process must be obtained from the washer-disinfector manufacturer.
- Perform automated cleaning and disinfection according to the operation manual of washer-disinfector.
- When cleaning the contra angle head with washer-disinfector, using a neutral enzymatic detergent (pH 7.0 - 8.0) is recommended.
- Immediately after cleaning, disinfecting and drying cycle is complete, the contra angle head must be removed from the washer-disinfector (within 1 hour) to prevent corrosion.
- After using the washer-disinfector, dry the product completely. Remaining moisture causes internal corrosion, etc.
- After cleaning and disinfecting the contra angle head, be sure to perform "7-3 Lubrication" and "7-4 Sterilization".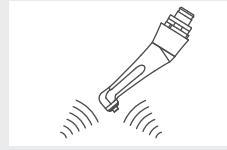


**[[For motor handpiece]]**

**⚠ CAUTION**


- The motor handpiece cannot be cleaned with washer-disinfector.

[Cleaning]

**1** Wipe the debris on the surface of the motor handpiece with a cloth moistened with water.

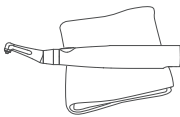
**2** Under appropriate lighting ( $\geq 500\text{lx}$ ), inspect the motor handpiece for contamination. If any visible contamination remains, repeat the process from procedure No. 1 until it is visually clean.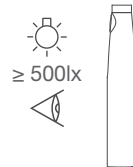

[Disinfection]


**1** Wipe the surface of the motor handpiece with disinfectant wipes (Minuten Wipes:ALPRO).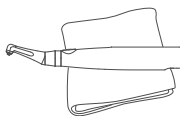

## Post-use Maintenance

### [Attach the motor cap]

To prevent dirt or liquid entering the motor handpiece, attach the motor cap to the motor handpiece when the contra angle head is not connected.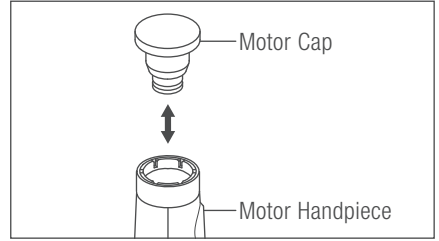

[[For stand]]

### [Cleaning]


Wipe the surface of the stand with a cloth moistened with water.


## 7-3 Lubrication

Lubricate the contra angle head using the F-type spray nozzle and the PANA SPRAY Plus.

- 1 Shake the PANA SPRAY Plus at least 2 or 3 times.
- 2 Screw the F-type spray nozzle into the PANA SPRAY Plus by turning it 10 times or so.

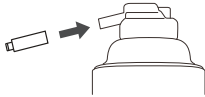
- 3 Attach the F-type spray nozzle to the rear end of the contra angle head.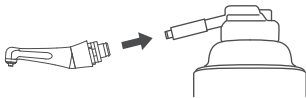

- 4 Spray until the oil coming out of the tip of the contra angle head is clear of any debris or foreign substance.

Spray time: 2 or 3 seconds  
Number of spray: 1 time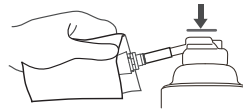

- 5 Remove the F-type spray nozzle from the contra angle head, and then remove the F-type spray nozzle from the PANA SPRAY Plus.
- 6 If excess oil comes out of the contra angle head, leave the contra angle head vertically for a while.

**▲ CAUTION**

- Be sure to use the PANA SPRAY Plus when lubricating the contra angle head.
- Lubricate only contra angle head. Do not lubricate products other than the contra angle head.
- Shake the PANA SPRAY Plus 2-3 times well before use.
- Be sure to securely connect the contra angle head to the F-type spray nozzle when lubricating the contra angle head. Otherwise, it may cause improper lubrication to the contra angle head and the oil may flow backward.

**7-4 Sterilization**

Use a steam sterilizer to sterilize the contra angle head. After each patient treatment, sterilize the contra angle head as follows.

Sterilizable equipment: contra angle head

- 1 Insert the contra angle head individually into a sterilization pouch that conforms to ISO 11607-1, and then seal the pouch.
- 2 Sterilize the contra angle head. Conduct sterilization as follows.

Type	Gravity Displacement	Pre-Vacuum (Dynamic Air Removal)
Temperature	132°C	134°C
Full Cycle Time	≥15min	3-18 min
Drying Time	≥30min	≥30min

### ⚠ CAUTION

- Follow local rules, regulations, and guidelines regarding the reprocessing of devices.
- Do not sterilize the motor handpiece.
- Do not sterilize the contra angle head with other instruments even when it is in a pouch. This is to prevent possible discoloration and damage to the contra angle head due to chemical residue on other instruments.
- Do not heat or cool the contra angle head quickly. Rapid change in temperature could cause damage to the contra angle head.
- Be sure to use sterilizers that can perform sterilization up to 135°C. In some sterilizers, the chamber temperature may exceed 135°C. Do not use these sterilizers as failure of the contra angle head could occur. Contact the sterilizer manufacturer for detailed information about cycle temperatures.
- Clean the contra angle head before disinfection. If blood remains on the internal surface it can become clotted and cause contra angle head failure.
- Keep the product in suitable atmospheric pressure, temperature, humidity, ventilation, and sunlight. The air should be free from dust, salt and sulphur.
- Do not touch the contra angle head immediately after autoclaving as it will be very hot and must remain in a sterile condition.
- Steam sterilization is recommended for the contra angle head. The validity of other sterilization methods (such as plasma sterilization or EOG sterilization) is not confirmed.
- Immediately after sterilization is complete (within 1 hour), remove the contra angle head from the sterilizer. Failure to do so may cause corrosion.

### NOTICE

- NSK recommends a Class B sterilizer as stated in EN 13060.
- For details, confirm the operation manual issued by manufacturer of the sterilizer.

## 7-5 Storage

Store the products in a clean place. Keep the contra angle head in a sterilization pouch until next use.

## 8 Maintenance


### 8-1 Changing the battery

iProphy uses a rechargeable battery. The battery pack has its life span (depending on the condition of use). If you feel that its charging time has become shorter or its operating time has become shorter, the battery pack may have come to the end of its life. In such a case, ask your authorized NSK dealer for battery pack change or purchase a new battery pack and replace the old one yourself. When replacing it yourself, be sure to observe the following "CAUTIONS ON CHANGING THE BATTERY." Note that NSK shall not be held liable for any malfunction or failure resulting from your not following the "CAUTIONS ON CHANGING THE BATTERY."


#### ▲ CAUTIONS ON CHANGING THE BATTERY

- Do not open any part other than the battery cover.
- Be sure to always use genuine NSK battery. Use of non-genuine battery can result in breakage, fluid leakage, rupture, or other problems.
- Do not work with wet hands. This may cause trouble due to short-circuiting of the battery and moisture infiltrating this product.
- Do not set the motor handpiece to the stand with the battery pack removed. This may cause damage to the product
- If the battery fluid leaks and gets in your eyes or on your skin or clothing, immediately wash your eyes and skin thoroughly with running water and see your doctor. Otherwise, may result in vision loss or skin complications.


- 1 Turn OFF the power of the motor handpiece.
- 2 Remove the rubber cover in the screw hole of the battery cover using a screwdriver.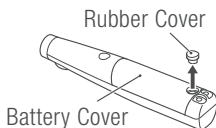


- 3 Remove the screw securing the battery cover using a screwdriver.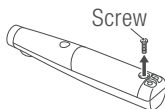

- 4 Remove the battery cover by lifting the rear (screw side) of the cover.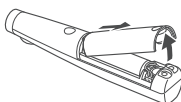

- 5 Take out the battery pack. Then hold the connector cords firmly and pull out the connector with force.

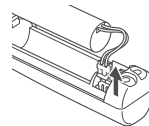
- 6 Prepare a new battery pack and straighten out the connector cords.


## Maintenance

- 7 Firmly insert the connector in the correct direction, following the instruction on the label.
- 8 Insert the battery pack into the motor handpiece.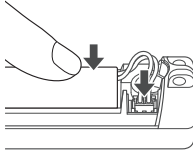

- 9 Tuck the extra cords in as shown below.

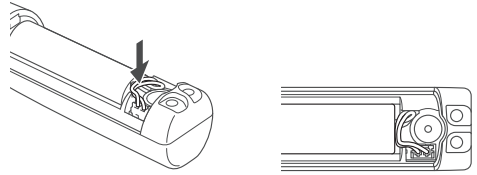### ⚠ CAUTION

- Make sure that the battery cords do not overlap with the screw hole as the battery cover cannot be attached correctly. Do not attach the battery cover forcibly as the cords may be broken.

- 10 Attach the battery cover to the motor handpiece with the screw.
- 11 Attach the rubber cover to the battery cover.
- 12 Check that the power can be turned ON.
- 13 Be sure to always charge the battery before use.


### ⚠ CAUTION

- Be careful not to lose the rubber cover and the screw after their removal.
- Never throw a battery pack into fire. It may rupture and cause an accident.
- For the disposal of used battery packs, see “10–4 Disposing product”.
- In case of difficulty in inserting the connector, the direction of the connector may be incorrect. Do not insert it by force, confirm the connector direction.

## 8-2 Mounting and Removing the AC Adaptor Plug Head

When removing the plug head from the AC adaptor, press the removal button and remove the adaptor head. When mounting the plug head to the AC adaptor, mount according following procedure.

- 1 Mount the plug head to the AC adaptor (1).
- 2 Slide the plug head to the direction of cord (2).

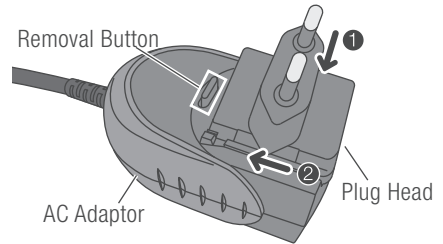## 8-3 Periodical Maintenance Checks

Perform periodical maintenance checks every three months, referring to the check sheet below. If any abnormalities are found, contact your authorized NSK dealer.

Points to check	Details
Rotation	Connect the contra angle head to the motor handpiece and insert the cup or brush to the contra angle head. After that, rotate the motor handpiece. Check for abnormalities such as abnormal rotation, vibration, noise, and overheating.
Charger	Set the motor handpiece to the stand and fully charge the battery. See "8-1 Changing the battery".

## 9 Troubleshooting

### 9-1 Error code

When the device stops due to failure, excessive pressure, disconnection and incorrect usage, the error LED lights up on the operation panel. When this happens, restart the motor handpiece to check if the error LED lights up. If there are no problems, the error is not displayed and the operation can be resumed. If the error is displayed again, refer to the table below and take appropriate actions.

Errors are displayed by the flashing of the Reverse Rotation Indicator LED and the lights of the Rotation Speed Indicator on the Operation Panel.

Error LED ● Light on ○ Light off ◎ Flashing		Type of error	Cause	Action
Reverse rotation	⑤ ④ ③ ② ①			
◎	⑤ ④ ③ ② ①	Self-check error.	Circuit failure.	Contact authorized NSK dealer for repair.
◎	⑤ ④ ③ ② ①	Excess current.	Motor handpiece is locked.	Remove load.
◎	⑤ ④ ③ ② ①	Excess voltage.	Circuit failure.	Contact authorized NSK dealer for repair.
◎	⑤ ④ ③ ② ①	Motor overheating.	Prolonged operation under heavy load.	Leave for some time to cool down before resuming.
◎	⑤ ④ ③ ② ①	Abnormal battery.	Abnormal battery or end-of-life battery.	Replace the battery pack.

Error LED ● Light on ○ Light off ◎ Flashing		Type of error	Cause	Action
Reverse rotation	⑤ ④ ③ ② ①			
◎	⑤ ④ ③ ② ①	Low battery voltage.	Insufficient charging or end-of-life battery.	Charge or replace the battery pack.
◎	⑤ ④ ③ ② ①	High battery voltage.	Battery or circuit failure.	Contact authorized NSK dealer for repair.
◎	⑤ ④ ③ ② ①	Outside the range of operating temperature.	Outside the operating temperature range or the disconnection of the thermistor of the battery.	Use it within the operating temperature range.
◎	⑤ ④ ③ ② ①	Overheated battery.	Overheated battery.	Charge the battery under the appropriate use environment (See "11-1 Specifications"). Replace the battery pack. If the problem is repeated with the new battery pack, it may be circuit failure, so contact authorized NSK dealer.

## Troubleshooting

### 9-2 Problems and actions

When trouble is found, check the following again before contacting your dealer. If none of these is applicable or the trouble is not remedied even after action has been taken, a failure of this product is suspected. Contact your authorized NSK dealer.

[Motor handpiece, stand, AC adaptor]

Problem	Cause	Action
The battery does not charge (Battery Indicator does not light or flash)	The AC adaptor is not connected to the stand.	Check the connection.
	The AC adaptor plug is not connected to an outlet, or there is no power supply from the outlet.	
	The motor handpiece is set on the stand in the wrong direction.	Set the motor handpiece into the stand in the correct direction.
	Poor connection between the stand and the motor handpiece.	Clean up the charging terminal.
	The battery pack is already full or nearly full.	There is no problem.
	The temperature of the battery pack is low.	No charging when the temperature of the battery pack is 0°C or less. Charge within the operating temperature range in the use environment (Watch out for condensation).
	The temperature of the battery pack is high.	No charging when the temperature of the battery pack is 40°C or higher. Lower the temperature of the use environment.

[Motor handpiece]

Problem	Cause	Action
Rotation of the motor handpiece is weak.	The remaining capacity of the battery pack is low.	Charge the battery. If there is no improvement after charging, replace the battery pack.
The maximum rotation speed of the motor handpiece is low.		
Motor handpiece does not rotate.	Foreign substance in the contra angle head.	Clean or replace the contra angle head.
	Failure inside the motor handpiece.	Contact authorized NSK dealer for repair.
	Short circuit in the ON/OFF key.	
	Battery voltage of the battery pack is low.	

## 10 After-sales Service

### 10-1 Warranty

NSK products are warranted against defects in manufacturing, workmanship and materials. NSK reserves the right to analyze and determine the cause of any problem. Warranty is voided should the product not be used in accordance with this manual or has been tampered with by unqualified personnel or has had non NSK parts installed. Replacement parts are available for seven years after discontinuation of the model.

### 10-2 Spare parts list

Model	REF	Remarks
iProphy Motor handpiece	EA11170020	-
AC ADAPTOR	Y1500014	POWER ADAPTOR
iProphy Stand	UA10910020	-
BATTERY PACK	E1080052	Rechargeable Li-ion Battery
F-Type Spray Nozzle	Z256090	-

### 10-3 Option parts list

[Contra angle head]

Model	REF	Gear Ratio	Remarks
iFX57	C1102	4:1 Reduction	<ul style="list-style-type: none"> <li>• For Prophy</li> <li>• screw-in type</li> </ul>
PR-F	H209		<ul style="list-style-type: none"> <li>• For Disposable Prophy Angle</li> </ul>

- Do not connect the contra angle head other than listed above.

### 10-4 Disposing product

In order to avoid the health risks of operators handling the disposal of medical equipment, as well as the risks of environmental contamination caused thereof, a surgeon or a dentist is required to confirm the equipment is sterile. Ask specialist firms who are licensed to dispose of specially controlled medical wastes, to dispose of the product for you.

The battery pack (Li-ion battery) included in the motor handpiece is a recyclable precious resource.

When disposing of it, do not disassemble the battery pack and dispose of the product according to laws and regulations.

Alternatively, return it to your authorized NSK dealer.

# 11 Specifications

## 11-1 Specifications

[Motor handpiece]

Model	iProphy
Rated input	DC3.7V 0.6A
Max. Rotation Speed (Contra angle head) (No load)	10,000 min <sup>-1</sup> *1
Continuous operation time (full charge)	Approx. 1 hour*2
Charging time	Approx. 1.5 hours
Dimensions	Ø25×D139mm
Weight	78g

\*1: Rotation speed may vary depending on the remaining capacity of the battery pack.

\*2: Operation time may vary depending on the load during use.

[Stand]


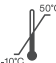
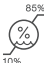
Model	NE293
Rated input	DC5V 0.6A
Dimensions	W80×D105×H37mm
Weight	120g

[AC adaptor]

Model	TR15RAM050
Rated input	AC100-240V 0.4 - 0.2A
Rated output	DC5V 2A
Dimensions	W48×D81×H43.3mm
Cord length	1.8m
Weight	140g

[Contra angle head]

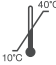Model	iFX57
Max. Rotation Speed (Motor handpiece)	10,000 min <sup>-1</sup>
Max. Rotation Speed (No load)	2,500 min <sup>-1</sup>
Gear Ratio	4:1 Reduction
Rotary Instrument Type	Screw-in cup / brush
Rotary Instrument Motion	360°Rotation
Weight	38g

	Temperature	Humidity	Pressure
Use Environment			-
Transportation and Store Environment			

\*No Condensation

## 11-2 Classification of equipment

- Type of protection against electric shock:
  - Class II equipment: 
- Degree of protection against electric shock:
  - Type B applied part: 
  - (Applied Part: Contra Angle Head, Motor Handpiece, Cup or Brush)
- Method of sterilization or disinfection recommended by the manufacturer:
  - See “7-4 Sterilization”
- Degree of safety of application in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide:
  - Equipment NOT suitable for use in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide.
- Mode of operation:
  - Intermittent operation (ON:3min OFF:5min)

## 11-3 Operation principle

By pressing the ON/OFF key on the motor handpiece, the motor is supplied with electric power and the brush or the cup attached to the contra angle head rotates.

## Specifications

### 11-4 Symbol

	TÜV Rheinland of North America is a Nationally Recognized Testing Laboratory (NRTL) in the United States and is accredited by the Standards Council of Canada to certify electro-medical products with Canadian National Standards.		Type B conforming component
	Follow the waste of electric and electronic equipment (WEEE) Directive (2012/19/EU) to dispose of the product and accessories.		Marking on the outside of Equipment or Equipment parts that include RF transmitters or that apply RF electromagnetic energy for diagnosis or treatment.
	Consult operation instructions		UL Recognized Component
	Manufacturer		Product Safety Electrical Appliance and Materials
	Class II equipment		Alternating current
	This conforms to CE European Directive of "Medical equipment directive 93/42/EEC."		Direct current (connection for power supply)
	Mandatory conformance mark on many products placed on the market in the European Economic Area		Directive 2011/65/EU of the European Parliament and of the Council of 8 June 2011 on the restriction of the use of certain hazardous substances in electrical and electronic equipment
	Catalog number		RoHS
	Date of manufacture		Polarity of d.c. power connector
	Authorised representative in the European community		INTERNATIONAL EFFICIENCY LEVEL V
	Serial No.		Temperature limitation
	Caution, Consult operation instructions		Atmospheric pressure limitation
	This product can be cleaned using a washer-disinfector.		Humidity limitation
	This product can be sterilized in a steam sterilizer up to Max. 135 °C. *For detail see "7-4 Sterilization"		fragile / handle with care
	This product can be sterilized in a steam sterilizer at 135°C. *For detail see "7-4 Sterilization"		This is the correct upright position of the distribution packages for transport and/or storage
			keep dry
			For indoor use only

## 12 EMC Information (Electromagnetic Compatibility Information)

Guidance and manufacturer's declaration - electromagnetic emissions		
The iProphy is intended for use in the electromagnetic environment specified below. The customer or the user of the iProphy should assure that is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR11/EN55011	Group 1 Class B	The iProphy uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR11/EN55011	Group 1 Class B	The iProphy is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC/EN 61000-3-2	Class A	
Voltage fluctuations/flicker emissions IEC/EN 61000-3-3	Complies	

Guidance and manufacturer's declaration - electromagnetic immunity			
The iProphy is intended for use in the electromagnetic environment specified below. The customer or the user of the iProphy should assure that it is used in such an environment.			
Immunity test	IEC60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC/EN 61000-4-2	$\pm(2,4)$ 6 kV contact $\pm(2,4)$ 8 kV air	$\pm(2,4)$ 6 kV contact $\pm(2,4)$ 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC/EN 61000-4-4	$\pm 2$ kV for power supply lines $\pm 1$ kV for input/output	$\pm 2$ kV for power supply lines $\pm 1$ kV for input/output	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC/EN 61000-4-5	$\pm 1$ kV line(s) to line(s) $\pm 2$ kV line(s) to earth	$\pm 1$ kV line(s) to line(s) $\pm 2$ kV line(s) to earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC/EN 61000-4-11	<5% Ut (>95% dip in Ut) for 0.5 cycle 40% Ut (60% dip in Ut) for 5 cycles 70% Ut (30% dip in Ut) for 0.5 cycles <5% Ut (>95% dip in Ut) for 5 sec	<5% Ut (>95% dip in Ut) for 0.5 cycle 40% Ut (60% dip in Ut) for 5 cycles 70% Ut (30% dip in Ut) for 25 cycles <5% Ut (>95% dip in Ut) for 5 sec	Mains power quality should be that of a typical commercial or hospital environment. If the user of the iProphy requires continued operation during power mains interruptions, it is recommended that the iProphy be powered from an uninterruptible power supply or a battery.
Power frequency (50/60Hz) magnetic field IEC/EN 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE: Ut is the a.c. mains voltage prior to application of the test level.			

## EMC Information (Electromagnetic Compatibility Information)

Guidance and manufacturer's declaration - electromagnetic immunity			
The iProphy is intended for use in the electromagnetic environment specified below. The customer or the user of the iProphy should assure that it is used in such an environment.			
Immunity test	IEC60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC/EN 61000-4-6	3Vrms 150 kHz to 80MHz	3 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the iProphy, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.
Radiated RF IEC/EN 61000-4-3	3V/m 80MHz to 2.5 GHz	3 V/m	<p>Recommended separation distance</p> $d = 1.2 \sqrt{P}$ $d = 1.2 \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = 2.3 \sqrt{P} \quad 800 \text{ MHz to } 2.5 \text{ GHz}$ <p>Where <math>P</math> is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and <math>d</math> is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 
<b>NOTE 1</b> At 80MHz and 800MHz, the higher frequency range applies.			
<b>NOTE 2</b> These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
<p>a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobiles radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the iProphy is used exceeds the applicable RF compliance level above, the iProphy should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the iProphy.</p> <p>b Over the frequency range 150kHz to 80MHz, field strengths should be less than 3 V/m.</p>			

## EMC Information (Electromagnetic Compatibility Information)

Cables and accessories	Maximum length	Complies with	
AC adaptor	1.8 m	RF emissions, CISPR11 , Harmonic emissions, Voltage fluctuations/ flicker emission Electrostatic discharge (ESD) Electrical fast transient/burst Surge Voltage dips, short interruptions and voltage variations on power supply input lines Power frequency (50/60Hz) magnetic field Conducted RF Radiated RF	Class B/ Group 1 IEC/EN 61000-3-2, Class A IEC/EN 61000-3-3 IEC/EN 61000-4-2 IEC/EN 61000-4-4 IEC/EN 61000-4-5 IEC/EN 61000-4-11  IEC/EN 61000-4-8 IEC/EN 61000-4-6 IEC/EN 61000-4-3

### Recommended separation distances between portable and mobile RF communications equipment and the iProphy

The iProphy is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the iProphy can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the iProphy as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter		
	150 kHz to 80 MHz $d = 1.2 \sqrt{P}$	80 MHz to 800 MHz $d = 1.2 \sqrt{P}$	800 MHz to 2.5 GHz $d = 1.2 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance  $d$  in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

**NOTE 1** At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

**NOTE 2** These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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NSK Europe GmbH